

DiaDent DiaPrep Pro is an EDTA cream with Urea Hydrogen Peroxide, used as an irrigation agent for chemomechanical preparation of the root canal.

Indications (use in conjunction with sodium hypochlorite)

- Debridement in the root canal
- Dissolution of pulp remnants
- Removal of smear layer
- Facile movements of canal instruments
- Microbicidal treatment within the canal
- Clearance of dentin tubules and apical area

Composition

- Ethylenediaminetetraacetic Acid (EDTA)
- Urea Hydrogen Peroxide (UHP)

Advantages and Effectiveness

- EDTA removes smear layer, softens dentin, and facilitates the removal of calcific obstruction
- UHP reacts with sodium hypochlorite to produce oxygen bubbles that flush out debris within the canal
- The lubricant medium facilitates instrument movement within the canal to avoid instrument binding or breakage
- Unclogged dentin tubules and apical area permit better penetration of root canal medicaments and fillings

Directions

1. Isolate the tooth to be treated with a rubber dam
2. Open and clear the pulp chamber in a conventional manner
3. Inject ample amount of DiaPrep Pro into the pulp chamber or deposit DiaPrep Pro directly on the canal shaping instrument
4. Shape the root canal in a conventional manner
5. Flush the root canal as needed with sodium hypochlorite solution
5. Repeat step 3 ~ 5 until the canal is shaped and cleaned as desired

Precautions

- For dental use only
- Avoid extrusion of the excess cream out of the apex
- Protective gloves are recommended
- In case of accidental contact with eyes or prolonged exposure to oral tissues, flush immediately with ample amount of water
- Do not use after expiry date

Storage

- Store in a dry and cool place
- Keep the cap tightly sealed after each use
- Shelf life is 2 years

Symbols



Use by

LOT

Batch Code



Consult Instructions for use



Caution

Temperature
Limitation

Manufacturer

EC REP

Authorised Representative
in the European Community

Warranty

DiaDent will replace product that is proven to be defective. DiaDent does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the product described.

Before using, it is the responsibility of the user to determine the suitability of the product for its intended use.

The user assumes all risk and liability in connection therewith.

DiaPrep Pro Kit

- 6g x 2 syringes
- 10 disposable tips

European Representative

DiaDent Europe B.V.
Antennestraat 70, 1322AS Almere, the Netherlands
Tel: 00 31 (0)36 549 8607 Fax: 00 31 (0)36 536 7317

DiaDent Group International

16, Osongsaengmyeong 4-ro, Osong-eup,
Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, 28161, Korea
Tel : 82-43-266-2315
Fax : 82-43-262-8658
www.diadent.co.kr
diadent@diadent.co.kr

DiaDent Group International Inc.

11-3871 North Fraser Way, Burnaby
B.C., Canada, V5J 5G6
Tel : 1-604-451-8851
Fax : 1-604-451-8865
www.diadent.com
diadent@diadent.com

DiaDent Europe B.V.

Antennestraat 70, 1322AS
Almere, the Netherlands
Tel : 31-36-549-8607
Fax : 31-36-536-7317
www.diadenturope.com
diadent@diadenturope.com